TOKUYAMA **M-Bond**

CLINICAL PROCEDURE GUIDE

– How to use TOKUYAMA **M-Bond** –

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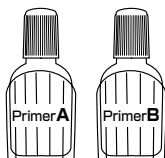
Mix & Apply
Primer A/B


Air-blow
after 30 sec.

Prosthesis/Restoration surface treatment

Non-precious metal	Precious metal	Ceramic/ Composite resin
Sandblast	Sandblast	Mix & Apply TOKUSO CERAMIC PRIMER [®] A/B
	Apply ^① METALTITE [®] and Dry	Air-blow after 10 sec.

※ ① Precious metal primer
※ ② Silane coupling agent

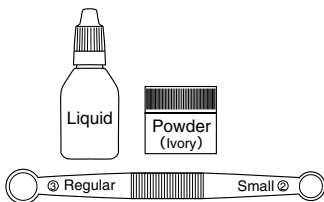Mixing Method


Regular : 3 drops of Liquid + 1 scoop of Powder (Regular side)

Small : 2 drops of Liquid + 1 scoop of Powder (Small side)

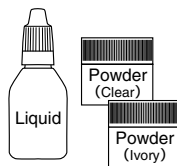Brush-on Method

Place adequate amount
of Liquid & Powder
on the Mixing Dish.

- ① Apply the cement ONLY to the prosthesis/restoration.
DO NOT apply the cement directly to the tooth surfaces or root canal.
- ② Remove the excess cement IMMEDIATELY after the setting of the prosthesis/restoration.

Clinical situations and the corresponding materials to be bonded

- Temporary fixation of a loose tooth

Case 1

Tooth to Tooth

- Cementation of orthodontic brackets

Case 2

Plastic/Non-Precious Metals

- Cementation of non-precious metal crowns, inlays, onlays and bridges including acid etch bridges

Case 3

Non-precious Metals

- Cementation of precious metal crowns, inlays, onlays and bridges including acid etch bridges
- Cementation of porcelain fused to precious metals
- Cementation of precious metal posts and cores

Case 4

Precious Metals

- Cementation of all ceramic and porcelain crowns, inlays, onlays, bridges and veneers

Case 5

Ceramics & Porcelains

- Cementation of cured composite crowns, inlays, onlays and bridges

Case 6

Cured Composites

- Repair of fractured porcelain fused to metal


Porcelain Repair

Porcelains

The Following illustration shows Cases and Materials to be bonded with compatible Primers.

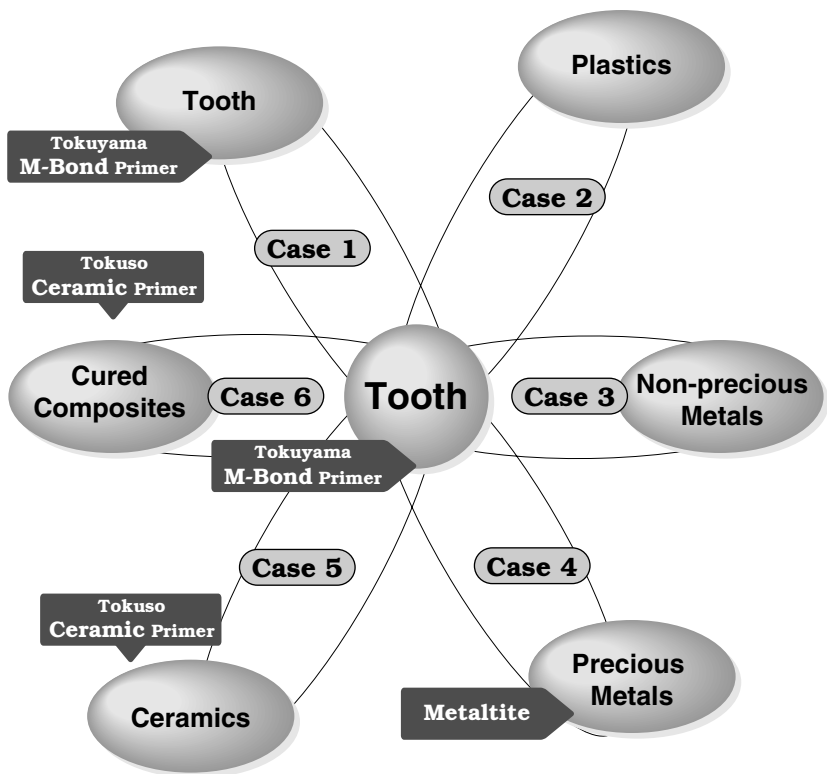(For detailed information, see Instruction for Use.)

Case 1

■ Temporary fixation of a loose tooth

Tooth Preparation

- **1** Mix Primers A and B
- **2** Apply mixed Primers, wait 30 seconds
- **3** Air dry gently (5 –10 seconds) ; **DO NOT RINSE**

Bonding

- **1** Place Liquid and Powder (Clear) into separate dimples of Mixing Dish.
- **2** Prepare the Disposable Brush for Brush-on technique (as noted in the instructions)
- **3** Dip Brush Tip into the Liquid
- **4** Touch the Brush Tip in the Powder
- **5** Place the drop of mixed cement on the surface to be bonded
- ▼ **6** Repeat steps 3-5 as needed

Case 2

■ Cementation of orthodontic brackets (plastic, non-precious metals)

Tooth Preparation

- **1** Mix Primers A and B
- **2** Apply mixed Primers, wait 30 seconds
- **3** Air dry gently (5 –10 seconds) ; **DO NOT RINSE**

Bonding

- **1** Place Liquid and Powder (Clear) into separate dimples of Mixing Dish
- **2** Prepare the Disposable Brush for Brush-on technique (as noted in the instructions)
- **3** Dip Brush Tip into the Liquid
- **4** Touch the Brush Tip in the Powder
- **5** Place the drop of mixed cement on the bracket surface to be bonded
- **6** Place the bracket to be bonded on the tooth surface

Case 3

- **Cementation of non-precious metal crowns, inlays, onlays and bridges including acid etch bridges**

Restoration Pretreatment

- [1] Sandblast
- [2] Clean Ultrasonically

Tooth Preparation

- [1] Mix Primers A and B
- [2] Apply mixed Primers, wait 30 seconds
- [3] Air dry gently (5 –10 seconds) ; **DO NOT RINSE**

Bonding

- [1] Mix Liquid and Powder
 - Regular: 3 drops of Liquid + 1 Scoop of Powder (Regular side)
 - Small: 2 drops of Liquid + 1 Scoop of Powder (Small side)
- [2] Seat restoration with mixed cement.
- [3] Remove excess cement.

REMARKS

1. DO NOT apply the mixed cement to tooth surface (especially into root canal); apply the mixed cement to prosthesis/restoration and then set to tooth (into root canal). Primer enhances the hardening of M-BOND.
2. Remove the surplus cement IMMEDIATELY after setting prosthesis/restoration. Intraoral setting time is designed “quick”.

Case 4

- Cementation of precious metal crowns, inlays, onlays and bridges including acid etch bridges
- Cementation of porcelain fused to precious metals
- Cementation of precious metal posts and cores

Restoration Pretreatment

- 1 Sandblast
- 2 Clean Ultrasonically
- 3 Apply METALTITE

Tooth Preparation

- 1 Mix Primers A and B
- 2 Apply mixed Primers, wait 30 seconds
- 3 Air dry gently (5 –10 seconds) ; **DO NOT RINSE**

Cementation and Final Bonding

- 1 Mix Liquid and Powder
 - Regular: 3 drops of Liquid + 1 Scoop of Powder (Regular side)
 - Small: 2 drops of Liquid + 1 Scoop of Powder (Small side)
- 2 Seat restoration with mixed cement.
- 3 Remove excess cement.

REMARKS

1. DO NOT apply the mixed cement to tooth surface (especially into root canal); apply the mixed cement to prosthesis/restoration and then set to tooth (into root canal). Primer enhances the hardening of M-BOND.
2. Remove the surplus cement IMMEDIATELY after setting prosthesis/restoration. Intraoral setting time is designed “quick”.

Case 5

- Cementation of all ceramic & porcelain crowns, inlays, bridges and veneers

Case 6

- Cementation of cured composite crowns, inlays onlays and bridges

Restoration Pretreatment

- **1** Sandblast
- **2** Clean ultrasonically
- **3** Mix TOKUSO CERAMIC PRIMER A and B
- ▼ **4** Apply the mixed TOKUSO CERAMIC PRIMER, wait 10 seconds

Tooth Preparation

- **1** Mix Primers A and B
- **2** Apply mixed Primers, wait 30 seconds
- ▼ **3** Air dry gently (5 –10 seconds) ; **DO NOT RINSE**

Cementation and Final Bonding

- **1** Mix Liquid and Powder
 - Regular: 3 drops of Liquid + 1 Scoop of Powder (Regular side)
 - Small: 2 drops of Liquid + 1 Scoop of Powder (Small side)
- **2** Seat restoration with mixed cement.
- ▼ **3** Remove excess cement.

REMARKS

1. DO NOT apply the mixed cement to tooth surface (especially into root canal); apply the mixed cement to prosthesis/restoration and then set to tooth (into root canal). Primer enhances the hardening of M-BOND.
2. Remove the surplus cement IMMEDIATELY after setting prosthesis/restoration. Intraoral setting time is designed “quick”.

Porcelain Repair

■ Repair of fractured porcelain fused to metals

Pretreatment

- 1 Sandblast or roughen with a diamond bur
- 2 Clean ultrasonically (Indirect Method*)
- 3 Mix TOKUSO CERAMIC PRIMER A and B
- 4 Apply the mixed TOKUSO CERAMIC PRIMER, wait 10 seconds
- 5 Apply METALTITE (where metal surface is exposed)

Cementation and Final Bonding

- 1 Mix Liquid and Powder
 - Regular: 3 drops of Liquid + 1 Scoop of Powder (Regular side)
 - Small: 2 drops of Liquid + 1 Scoop of Powder (Small side)
- 2 Seat restoration with mixed cement.
- 3 Remove excess cement.

*This procedure is indicated when the porcelain is not cemented to the abutment tooth.
If the porcelain is in the mouth, clean the prepared surface with a non-oily flour pumice, rinse with water, dry then apply TOKUSO CERAMIC PRIMER.

Specifications are subject to change without notice and without obligation on the part of the manufacturer.

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Manufacturer: Tokuyama Dental Corp.
EC Rep.: Tokuyama Europe GmbH
North/South America/Middle East: J. MORITA USA, Inc.