Primer for bonding precious metal to resin  
 Instructions for use  
 Primer zur adhäsiven Verbindung von Edelmetall mit Kompositwerkstoffen  
 Gebrauchsanleitung  
 Primaire d'adhésion - métaux précieux et résine  
 Mode d'emploi  
 Primer per l'adesione del metallo prezioso alla resina  
 Istruzioni di uso  
 Adhesivo para la unión de Metales Preciosos a Resina  
 Instrucciones de uso

# METALTITE

CE 0123  
 REF BIS 01/02

Manufacturer

## Tokuyama Dental Corporation

38-9 Taitou 1-chome, Taitou-ku, Tokyo, Japan

Tel: +81-3-3835-7201

Email: export@tokuyama-dental.co.jp

EC Rep.

## Tokuyama Europe GmbH

Oststrasse 10, 40211 Düsseldorf, Germany

Tel: +49 211 1754480 Fax: +49 211 357379

North America

## Tokuyama America, Inc.

California

Tel: (650) 571 8872

Tel: (877) 378 3548 (Dental Toll-Free)

## ENGLISH

### PRODUCT DESCRIPTION AND GENERAL INFORMATION

METALTITE is a primer to improve the adhesion between precious metals and resins.

One of the components of METALTITE is MTU-6, a thiouracil monomer, which produces a tenacious chemical bond between various resins and precious metals.

### CONTENT

METALTITE.....5mL

### CLINICAL APPLICATIONS

1. Pretreatment of precious metal alloys in preparation for bonding with polymerizable resin materials such as adhesive resin cements.
2. METALTITE can be used as a primer for the following procedures:
  - 1) Cementation of precious metal inlays, onlays, crowns, bridges etc.
  - 2) Cementation of precious metal post and cores.
  - 3) Repair of precious metal bonded porcelain in the mouth.
  - 4) Repair of precious metal based dentures before acrylic is applied on the broken area.
  - 5) Bonding of opaque resin to a precious metal base in fabricating a resin face crown indirectly or directly in the mouth.

### CLINICAL PROCEDURES

1. Roughen the surface of the restoration by sandblasting or grinding with a green stone. When using the sandblast procedure, use at least 30 to 50 micron grid of alumina powder.
2. Rinse and dry the surface.
3. Apply METALTITE on the surface to be bonded.  
 NOTE: One or two applications is sufficient. Too thick a layer may result in poor adhesion.  
 Blow drying is not necessary. The solvent of METALTITE evaporates rapidly.  
 DO NOT let saliva, blood, grease, water etc. come in contact with the surface to be bonded.  
 If the surface is contaminated, wipe the surface with an alcohol soaked mini sponge or cotton swab to remove contaminants. There is no need to reapply METALTITE as it cannot be removed with alcohol.
4. Bond the surface with resin of choice according to the manufacturer's instruction for the resin material used. Cover the surface as quickly as possible, after the surface has been covered with METALTITE.

### PRECAUTIONS

1. Use METALTITE only as directed. DO NOT use METALTITE for any other application.
2. METALTITE is intended for use only by licensed dental care professionals.
3. DO NOT use this product if the safety seals are broken or if the contents have been tampered with.
4. Always determine whether your patient is allergic or hypersensitive to methacrylic related monomers. DO NOT use METALTITE if a patient is allergic to or hypersensitive to these monomers. If METALTITE causes an allergic reaction or hypersensitivity, discontinue its use immediately.
5. Use examination gloves (plastic, vinyl, latex) at all times while using METALTITE to avoid the possibility of allergic reactions from methacrylic monomers. If METALTITE comes in contact with the examination gloves, immediately dispose of the examination gloves and wash hands thoroughly with water.
6. AVOID CONTACT WITH EYES. In case of contact with eyes, thoroughly flush eyes with water and immediately contact a physician.
7. In the event that METALTITE contacts the skin, clean the skin immediately with an alcohol soaked cotton swab or gauze.
8. METALTITE should not be ingested or aspirated. Ingestion or aspiration may cause serious injury.
9. To avoid cross infection, DO NOT reuse mini sponges. Also clean the mixing well with alcohol after use.
10. DO NOT leave METALTITE within the reach of patients and children.
11. AVOID open flames when using or disposing primers.

### STORAGE

1. Store at 0-25°C (32-77°F).
2. DO NOT use METALTITE after the indicated date of expiration.

### DISPOSAL

The remaining liquid should be absorbed in an inert absorbent material such as gauze or cotton, and should be disposed of in accordance with local regulations.

\*The manufacturer is not responsible for damage or injury caused by improper use of this product. It is the personal responsibility of the user to ensure the product is suitable for an appropriate application before use.  
 Specifications are subjected to change without notification.

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