

Step-by-Step Guide for using the Tokuyama Silicone Remover



Brought to you by Tokuyama Dental America




Step 1:

Create an incision with a flat tool (e.g. a scalpel or spatula) between the silicone and the acrylic denture on any edge of the denture.

Step 2:

Rotate the denture to a vertical position, making sure the side with the incision is at the top.

Keeping the incision open between the silicone and the acrylic denture with the flat tool, place a few drops of Tokuyama's Silicone Remover in the gap.

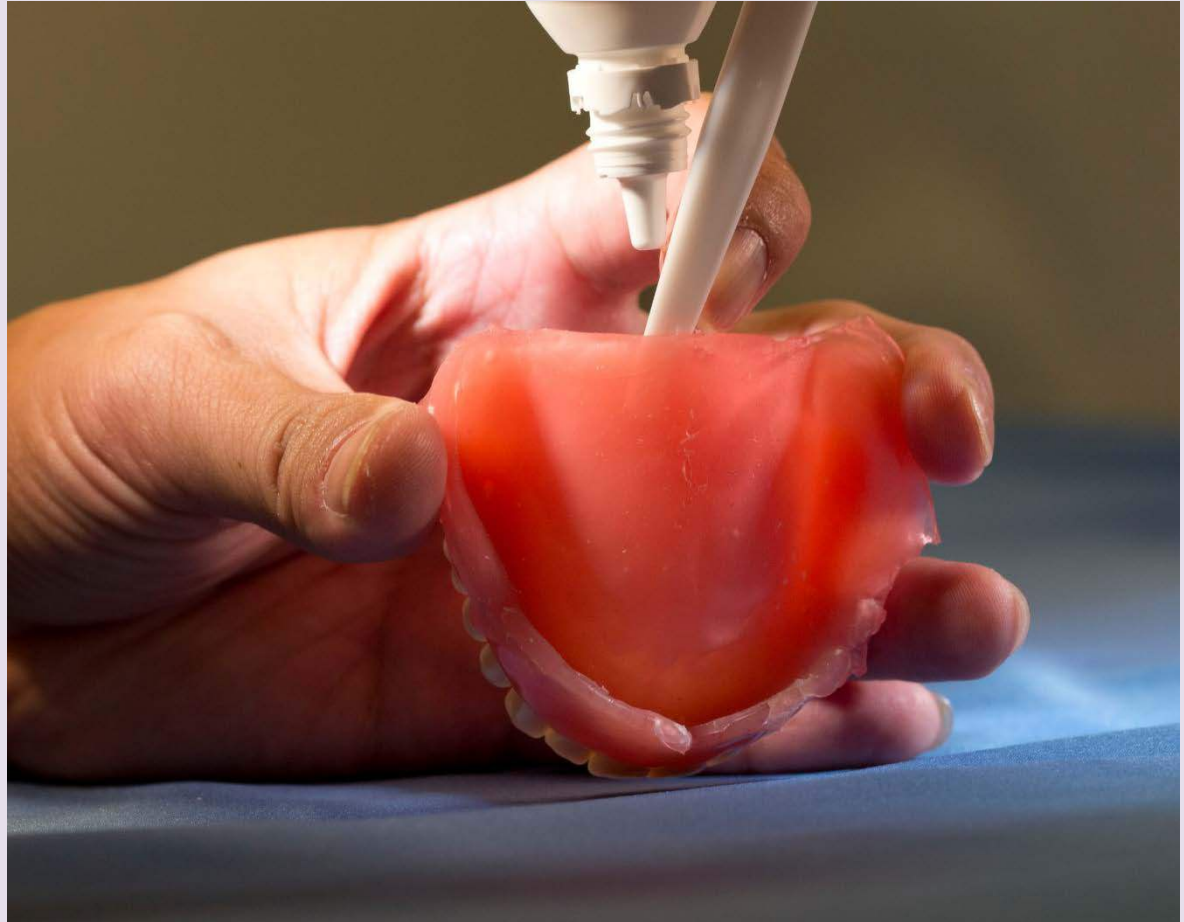



Step 3:


Remove the scalpel or flat tool from the incision and slowly begin to pull the silicone away.



Step 4:

As you continue to pull the silicone away, rotate the denture in a circular motion to spread the liquid.

The rotating motion guides the Silicone Remover to slowly delaminate the relined material.

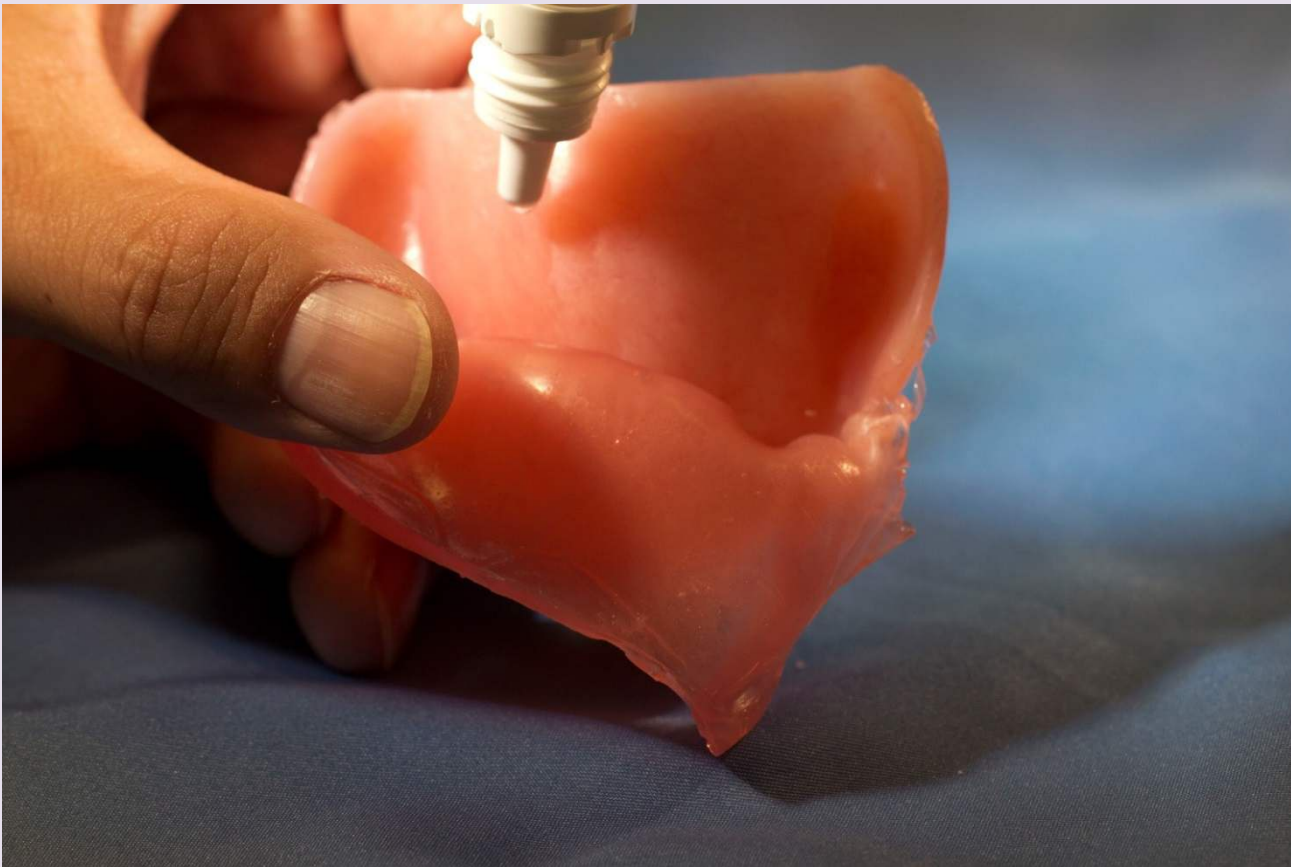

Step 5:

As needed, add a few drops of Silicone Remover and continue to slowly peel away the relined material. Remember to rotate the denture to spread the liquid and, if needed, use a flat tool to re-separate the silicone from the acrylic.

Congratulations!
You're all set to reline the denture.

